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The screenshot shows the Orthosis Configurator interface. At the top left is the logo with the text "Orthosis Configurator". To the right are links for "新規構成", "JA", "FAQ/サポート", and "サインイン". Below the logo is a progress bar with four steps, where step 1 is active. A button labeled "現在の構成を表示" is below the progress bar. On the right side of the progress bar, a button labeled "続ける" is highlighted with a red box. Below the progress bar, the heading "会社データ" is followed by the instruction "選択してください。". The main content area is divided into three columns. The first column, titled "国", contains a dropdown menu with "日本" selected and highlighted with a red dashed box, and the text "国を選択してください。". The second column, titled "テクニカルサポート", contains the text "Matsumoto P&O Co., Ltd.", "電話: +81 568 47 1701", and "メール: info@pomgs.co.jp". The third column, titled "販売", contains the text "Matsumoto P&O Co., Ltd.", "210-3 Ooaza-Hayashi", "485-8555 Komaki-City (日本)", and "メール: info@pomgs.co.jp".

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The screenshot shows the 'Orthosis Configurator' web application. The top navigation bar includes '新規構成' (New Configuration), 'JA' (Japanese), 'FAQ/サポート' (FAQ/Support), and 'サインイン' (Sign In). Below the navigation bar is a progress indicator with steps 1, 2, 3, and 4. Step 1 is active, and a '現在の構成を表示' (Show current configuration) button is visible. A '続ける' (Continue) button is highlighted with a red box. The main content area is titled '靴と上側足関節の測定' (Shoe and Dorsal Ankle Measurement). It contains two main sections: '靴のサイズ (EU式) *' (Shoe Size (EU Style) *) with a dropdown menu set to '32', and '靴底と補償測定' (Shoe Sole and Compensation Measurement) with a diagram of a shoe and a table for '高さ補償 (C)' (Height Compensation (C)) and '踵高さ (A)' (Heel Height (A)). The table has two rows: '高さ補償 (C)' with a dropdown set to '0 - 5 mm' and '踵高さ (A)' with a dropdown set to '0 - 10 mm'. To the right, '上側足関節の運動範囲' (Dorsal Ankle Range of Motion) shows a diagram of an ankle and two dropdown menus: '背側' (Dorsal) set to '35 °' and '底側' (Plantar) set to '20 °'. A note at the bottom right says '選択してください。 | * 必須フィールド' (Please select. | * Required field).

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The screenshot shows the 'Orthosis Configurator' interface. At the top, there are navigation links for '新規構成' (New Configuration), 'JA' (Japanese), 'FAQ/サポート' (FAQ/Support), and 'サインイン' (Sign In). Below this is a progress bar with four steps, where step 1 is currently selected. The main content area is titled '筋肉強度' (Muscle Strength) and includes a sub-header 'Jandaによる分類' (Classification by Janda). A central diagram of a human leg is surrounded by six dropdown menus for selecting muscle strength levels: '股関節屈曲' (Hip Flexion) set to 5, '股関節伸展' (Hip Extension) set to 4, '膝関節伸展' (Knee Extension) set to 4, '膝関節屈曲' (Knee Flexion) set to 4, '背屈' (Dorsiflexion) set to 4, and '底屈' (Plantar Flexion) set to 1. To the right of the diagram is a '筋肉強度の説明' (Muscle Strength Explanation) section, which lists the five strength levels with their corresponding descriptions in Japanese. A red box highlights the dropdown menus and the central diagram.

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The screenshot shows the 'Orthosis Configurator' interface. At the top, there is a logo and navigation links for '新規構成', 'JA', 'FAQ/サポート', and 'サインイン'. Below this is a progress bar with four steps: 1 (selected), 2, 3, and 4. A '戻る' (Back) button is on the left, and a '続ける' (Continue) button is on the right, highlighted with a red box. Below the progress bar, the text '活動レベル' (Activity Level) is displayed, followed by the instruction '選択してください。' (Please select). There are four icons representing different activity levels: 1. 屋内歩行 (Indoor walking), 2. ある程度の屋外歩行 (Moderate outdoor walking), 3. 無制限の屋外歩行 (Unlimited outdoor walking), and 4. 特に高強度の無制限屋外歩行 (Especially high-intensity unlimited outdoor walking). The third icon has a checkmark, indicating it is the selected option.

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The screenshot shows the 'Orthosis Configurator' interface. At the top, there are navigation links for '新規構成' (New Configuration), 'JA' (Japanese), 'FAQ/サポート' (FAQ/Support), and 'サインイン' (Sign In). Below this is a progress bar with four steps, where step 2 is active. A '戻る' (Back) button is on the left, and a '続ける' (Continue) button is on the right, highlighted with a red box. The main section is titled '足関節機能 - 腹側' (Ankle Joint Function - Dorsal). It contains four selection cards: '背屈ストッパーなし' (No Dorsiflexion Stopper), '静的背屈ストッパー' (Static Dorsiflexion Stopper), '動的背屈ストッパー' (Dynamic Dorsiflexion Stopper), and 'システム足関節なし' (No System Ankle Joint). The '動的背屈ストッパー' card is selected with a checkmark. Below this is the '腹側ばねユニット (背屈に対抗する)' (Dorsal Spring Unit (Resisting Dorsiflexion)) section, which shows five spring options with different stiffness levels and maximum ranges of motion: 'ばね力 ノーマル (青)' (Normal Spring Force - Blue, 15°), 'ばね力 中 (緑)' (Medium Spring Force - Green, 15°), 'ばね力 強い (白)' (Strong Spring Force - White, 10°), 'ばね力 非常に強い (黄)' (Very Strong Spring Force - Yellow, 10°), and 'ばね力 特別に強い (赤)' (Especially Strong Spring Force - Red, 5°). The '非常に強い (黄)' option is selected with a checkmark. A legend at the top of the selection area indicates: Green for '推奨' (Recommended), Yellow for '代替' (Alternative), Red for '機能関連の除外' (Exclusion related to function), and Grey for '機能/荷重関連の除外' (Exclusion related to function/load).

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The screenshot shows the 'Orthosis Configurator' web application. At the top, there is a navigation bar with the logo, language selection (JA), and links for '新規構成', 'FAQ/サポート', and 'サインイン'. Below the navigation bar is a progress indicator with four steps (1, 2, 3, 4) and a '現在の構成を表示' button. A red box highlights the '続ける' (Continue) button. The main content area is titled '装具シェル' and includes a legend with four categories: '推奨' (Recommended, green), '代替' (Alternative, yellow), '機能関連の除外' (Exclusion of function-related, red), and '機能/荷重関連の除外' (Exclusion of function/load-related, grey). The '下腿シェル' (Lower leg shell) section shows a 3D model of a lower leg and two options: a front shell (highlighted in green) and a back shell (highlighted in yellow). The 'フットピース' (Foot piece) section shows a 3D model of a foot and three options: a red foot piece (highlighted in red), a green foot piece (highlighted in green), and a yellow foot piece (highlighted in yellow). Red arrows point to the sole of the foot model, indicating the stiffness of the foot piece.

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The screenshot shows the 'Orthosis Configurator' interface. At the top, there are navigation links for '新規構成' (New Configuration), 'JA' (Japanese), 'FAQ/サポート' (FAQ/Support), and 'サインイン' (Sign In). Below this is a progress bar with four steps, where step 2 is active. A '現在の構成を表示' (Show current configuration) button is visible. The main content area is titled '足関節機能 - 背側' (Ankle Joint Function - Dorsal) and includes a legend with four categories: '推奨' (Recommended) in green, '代替' (Alternative) in yellow, '機能関連の除外' (Exclusion of related functions) in red, and '機能/荷重関連の除外' (Exclusion of function/load related) in grey. Four configuration options are shown in boxes, each with a plus sign and an information icon:

- Option 1 (Yellow border): 底屈ストッパーなし (No plantar flexion stopper).
- Option 2 (Red border): 底屈ストッパーなし、背屈支援 (No plantar flexion stopper, plantar flexion support).
- Option 3 (Green border, selected): 動的底屈ストッパー、背屈支援 (Dynamic plantar flexion stopper, plantar flexion support).
- Option 4 (Yellow border): 静的底屈ストッパー (Static plantar flexion stopper).

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The screenshot shows the 'Orthosis Configurator' interface. At the top, there are navigation links for '新規構成' (New Configuration), 'JA' (Japanese), 'FAQ/サポート' (FAQ/Support), and 'サインイン' (Sign In). Below this is a progress bar with four steps: 1 (selected), 2, 3, and 4. A '戻る' (Back) button is on the left, and a '続ける' (Continue) button is on the right, highlighted with a red box. The main content area is titled '足関節システム' (Ankle System) and features a legend with four categories: '推奨' (Recommended) in green, '代替' (Alternative) in yellow, '機能関連の除外' (Exclusion related to function) in red, and '機能/荷重関連の除外' (Exclusion related to function/load) in grey. Below the legend are four product images in a grid, each with a 'pluqego' logo and a circled 'i' icon. The first and second images are enclosed in a red border, indicating they are excluded from the current configuration. The third and fourth images are enclosed in a green border, indicating they are recommended.

ID: 2021.11.01-0308969-R

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The screenshot shows the 'Orthosis Configurator' web application. At the top, there is a navigation bar with the logo, 'Orthosis Configurator', and links for '新規構成', 'JA', 'FAQ/サポート', and 'サインイン'. Below this is a progress indicator with steps 1, 2, 3, and 4. Step 2 is active, and there are buttons for '戻る' (Back), '現在の構成を表示' (Show current configuration), and '続ける' (Continue). The main content area is titled '足首高さの構造' (Ankle height structure) and includes a legend: green for '推奨' (Recommended), yellow for '代替' (Alternative), red for '機能関連の除外' (Exclusion of function), and grey for '機能/荷重関連の除外' (Exclusion of function/load). Three options are shown: '片製' (Single-sided) in a green box, '関節サポート付き' (With joint support) in a yellow box, and '両側' (Double-sided) in a red box. Below this is the '足関節システム幅' (Ankle joint system width) section, with a header for widths: 10 mm, 12 mm, 14 mm, 16 mm, and 20 mm. Five 'NEURO SWING' device options are shown, with the 12 mm and 14 mm versions highlighted in green and yellow boxes respectively.

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The screenshot shows the 'Orthosis Configurator' interface. At the top, there are navigation links for '新規構成' (New Configuration), 'JA' (Japanese), 'FAQ/サポート' (FAQ/Support), and 'サインイン' (Sign In). Below this is a progress bar with four steps, where step 2 is active. A '戻る' (Back) button is on the left, and a '続ける' (Continue) button is on the right, highlighted with a red box. The main section is titled '背側ばねユニット (底屈に対抗する)' (Dorsal Spring Unit (Resisting Flexion)). It contains five options, each with a spring icon and a red arrow indicating force. The options are: 1. 'ばね力 ノーマル(青)' (Spring Force Normal (Blue)) with a maximum range of 15°. 2. 'ばね力 中(緑)' (Spring Force Medium (Green)) with a maximum range of 15°, marked with a checkmark and a green border. 3. 'ばね力 強い(白)' (Spring Force Strong (White)) with a maximum range of 10°. 4. 'ばね力 非常に強い(黄)' (Spring Force Very Strong (Yellow)) with a maximum range of 10°. 5. 'ばね力 特別に強い(赤)' (Spring Force Especially Strong (Red)) with a maximum range of 5°. A legend at the top right indicates: green for '推奨' (Recommended), yellow for '代替' (Alternative), red for '機能関連の除外' (Exclusion related to function), and grey for '機能/荷重関連の除外' (Exclusion related to function/load). A note says '選択してください。' (Please select).

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The screenshot shows the Orthosis Configurator interface. At the top, there is a navigation bar with the logo, language selection (JA), and links for '新規構成', 'FAQ/サポート', and 'サインイン'. Below this is a progress indicator with four steps, where step 3 is active. A '続ける' (Continue) button is highlighted with a red box. The main content area is titled '個別調整' (Individual Adjustment) and includes a 3D model of a leg with an ankle joint. To the right of the model are dropdown menus for '足関節 - 外側', '足関節 - 内側', '膝関節 - 外側', and '膝関節 - 内側'. The selected 'NEURO SWING system ankle joint' configuration is shown in a box with the following settings:

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